


Product specification

Fisscure is a manually operated mechanical device consisting of a conical-pointed cylindrical tube, equipped with an internal action mechanism necessary for the release of 4 needles from the cylinder surface perpendicular to the longitudinal axis of the device, separated by an angular distance of 90°. The mechanism is operated by the doctor from the terminal part of the device, where there is a knob that can be turned along the axis of the device, which has a safety button for releasing or locking the internal action mechanism. The outer cylinder is equipped with measuring lines that allow to safely insert the device at the depth necessary to perform the treatment at the right point in the anal channel.

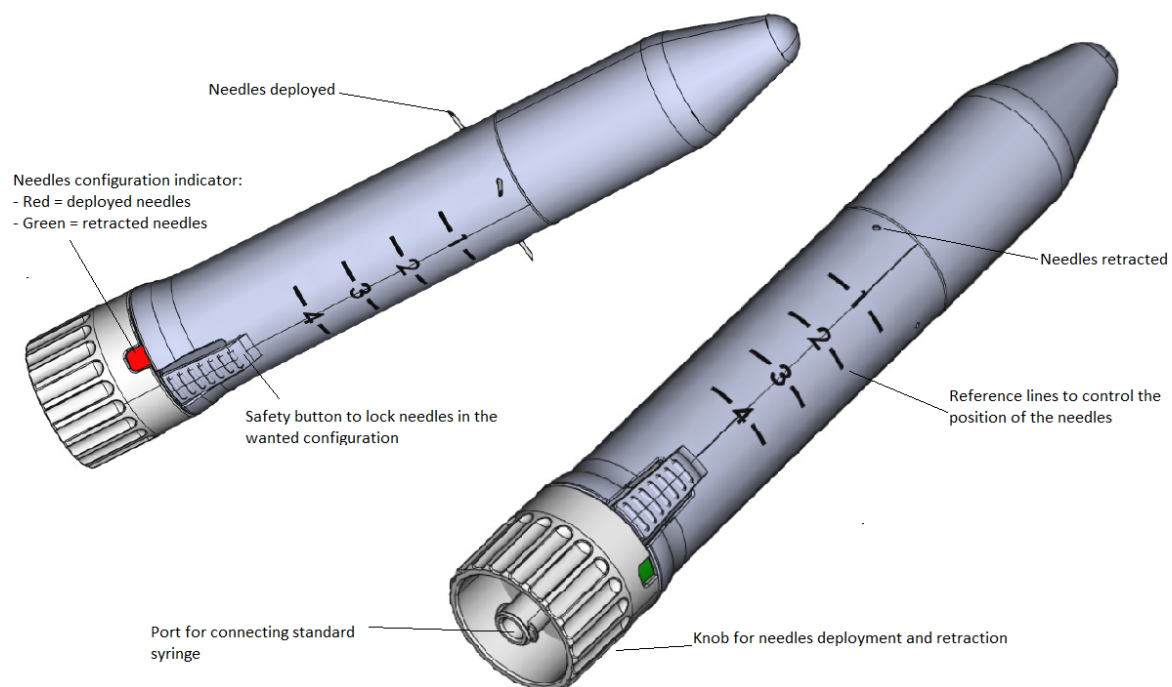

Attribute	Description
Length	152 mm
Diameter body	22,5 mm
Diameter handle	25,5 mm
Weight	27,5 gr
Syringe connectivity	Standard luerlock
Materials in contact with the patient	ABS/PC polymers blend (Bayblend M850 XF) coloured with Mevopur-Blue SB5M176382-ZA, stainless steel AISI 304, PLT 34 – 165 (black ink on the mantle)
Materials in contact with the drug to be applied	TPE (Chemiton Med TMB0NA80.001), MABS (Terlux 2822HD), stainless steel AISI 304 and Mevopur-Blue SB5M176382-ZA (handle color)
Needle	Bevel type: lancet short; material: AISI304; inner diameter: 0,40 mm; outer diameter: 0,60 mm
Sterility status	Device delivered in sterile condition after Ethylene Oxide sterilisation
Packaging	PET blister sealed with a Tyvek layer